



INSTRUCTIONS FOR USE

Versa 100 ml canister
Part Number: CPC-100

USE

The canister collection system is used to collect exudate and debris from the wound base.

INDICATIONS

Use canisters in conjunction with the Versa NPWT System. The 100 ml canister should be utilized for the collection of exudate from the wound.

CONTRAINDICATIONS

Do not use the Versa 100ml canister on any other NPWT device.

WARNINGS/CAUTIONS

- Read and understand all warnings, cautions, and directions completely and carefully before use. Failure to do so may result in serious injury.
- Do not reuse or reprocess this product. This device is for single use only. Reuse or reprocessing may damage the product which may result in injury, infection, or death.
- Do not use it if the individual package is opened or if the package is damaged. Carefully inspect the canister for damage or defects prior to use.
- Disposal of used canisters should follow facility policies or local ordinances relating to the handling of potentially infected or bio-hazardous materials.

INSTRUCTIONS FOR USE

1. Carefully read the WARNINGS and CAUTIONS prior to use.
2. Inspect the packaging for damage and defects. Replace the unit if the packaging is damaged or expired.
3. Verify physician's order for NPWT.
4. Open the canister packaging.
5. Clamp canister tubing.
6. Hinge canister in place on the back of the Versa device.
7. Ensure that you hear a 'click'.
8. Gently tug on the canister to ensure it is latched in place.
9. Connect canister tubing to port pad tubing.
10. Ensure canister clamp and port pad tubing clamp are unclamped.
11. Power on the Versa device.
12. Ensure seal integrity – dressing should have wrinkle appearance.

STORAGE AND TRANSPORT CONDITIONS

Storage Temperature: -30°C (-22°F) without relative humidity control to 60°C (158°F) up to 90% relative humidity (non-condensing). The storage and shipping conditions apply to the Versa 100 ml canister.



This product is non-sterile



Single use only



Do not use if package is damaged



Indicates the date of expiration



Indicates the manufacture batch code



Latex Free

RX Only

Caution: Federal U.S.A. law restricts this device to sale or use by or on the order of a physician (or properly licensed practitioner).



Manufactured by:
Cork Medical
8000 Castleway Drive
Indianapolis, IN 46250 USA