




Manufactured By:
Cork Medical
8000 Castleway Drive
Indianapolis, IN 46250
1-866-904-2946

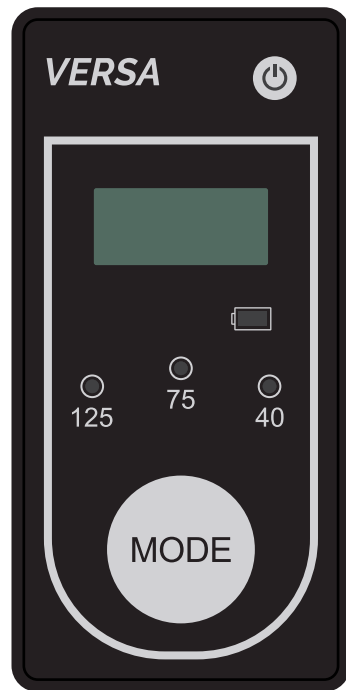

VERSA Quick Reference Guide

Important Safety Information
accompanies this device.

VERSA Quick Reference Guide

Before Using the VERSA NPWT System

- 1 Review the user manual with your doctor or nurse.
- 2 Review this quick reference guide.
- 3 Review the safety information sheet with your doctor or nurse.
- 4 Keep this quick reference guide and the safety information sheet in the pocket on the therapy unit carrying case.

Precautions

1. Do not use it while bathing or store where it can fall into standing water.
2. Do not retrieve the device in the event it has fallen into water.
3. Notify treating practitioner for unexpected fluid volume loss.
4. Never use expired supplies
5. Clamp and disconnect tubing prior to bathing.
6. If any liquids or food is spilled on pump, immediately wipe clean.
7. If NPWT is off for 2 or more hours, contact the medical provider immediately.
8. If bright red blood is visible in the drainage tubing or canister, turn off NPWT, leave dressing intact, and contact EMS immediately.
9. For medical questions, please call your healthcare provider or Cork Medical at 1-866-904-2946. In case of an emergency, call your local emergency service at (i.e. 911) immediately.

Cleaning Steps

- 1 Cork Medical recommends using a bacteriostatic cleaning wipe. (e.g., Clorox Wipes)
- 2 Turn pump off prior to cleaning the device.
- 3 Wipe all external surfaces of the pump with the bacteriostatic cleaning wipe.
- 4 Perform a visible inspection post cleaning and if any dirt, dust, lint and/or debris is present, repeat the cleaning process.
- 5 If any damage is noted, please contact your distributor or medical provider.

Power On/Off the VERSA NPWT System


1. Power on the Versa NPWT System by pressing the POWER button on the keypad. Located in the top right corner.
2. The pump will revert to previous settings set by your medical provider.
3. If any alarms occur see troubleshooting setting in IFU.
4. To power off the pump, press and hold the power button for 3 seconds, a countdown will occur from 3 when the power sequence begins.

Power Button



Attaching the Drainage Canister to the Pump


1. Attach the canister to the back of the Versa NPWT System by hooking the canister onto the pump hinge post then rotating to lock the canister into place.
2. The canister is snapped into place and will be secure once properly attached by the user. Gently tug on the canister to ensure it is secured.
3. The canister includes drainage tubing with a luer fitting. Attach the drainage tubing from the canister to the drainage tubing from the wound dressing kit.
4. Once the canister is in place and the drainage tubing is connected to the wound dressing kit, the Versa NPWT System may then be powered on.

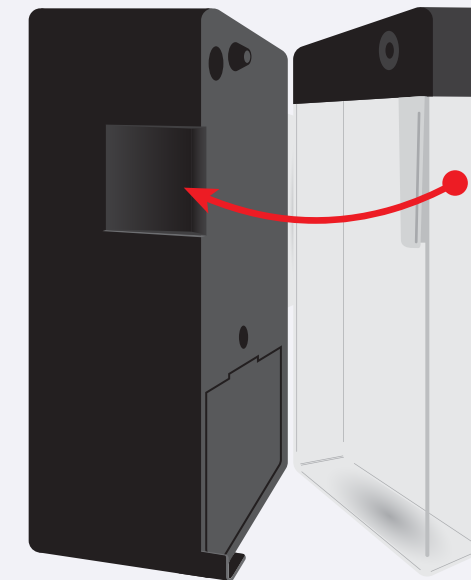

Use only the Cork Medical Versa NPWT 100ml canister with the Versa NPWT System.

System Disconnection/Reconnection

1. Clamp canister tubing.
2. Clamp port pad tubing, the tubing that is connected to the wound.
3. Locate the connection port between the canister tubing and the port pad tubing.
4. Unscrew counterclockwise.
5. Cover the end of port pad tubing and canister tubing with a medical glove.
6. Perform activities needed.
7. To reconnect, connect the port pad tubing to the canister tubing, and turn Leur lock clockwise. (Don't Overtighten)
8. Unclamp port pad tubing.
9. Unclamp canister tubing.
10. Ensure the dressing is pulled down, will have a raisin like appearance.

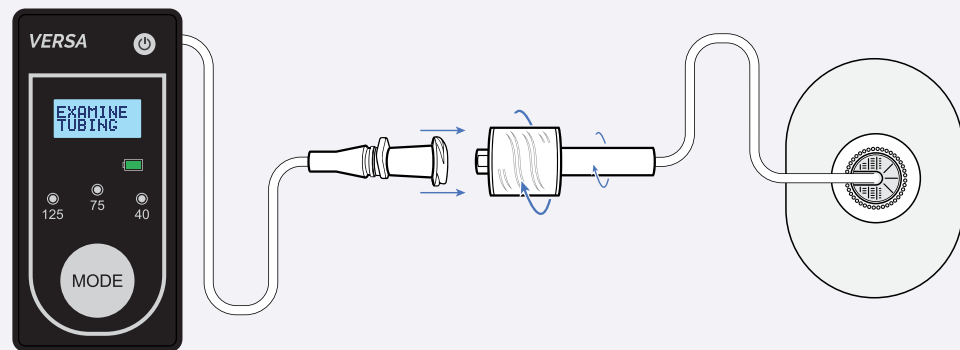

Battery Critical Alarm



Alarm Display

Visual message displayed along with audible alarm.

If alarm not corrected, continuous audible alarm will occur.



Troubleshooting

Battery life is critically low.

Turn off pump and Replace 3-AA batteries and dispose of them accordingly, Lithium batteries are recommended.

! If troubleshooting fails to rectify alarm, contact your medical provider.

Canister Full Alarm



Alarm Display

Visual message displayed along with audible alarm.

If alarm not corrected, continuous audible alarm will occur.



Troubleshooting

Collection canister full.

Change canister to resolve alarm.

! If troubleshooting fails to rectify alarm, contact your medical provider.

Leak Alarm



Alarm Display

Visual message displayed along with audible alarm.

If the alarm is not corrected, continuous audible alarm will occur.



Troubleshooting

Drainage tubing not connected.

Wound dressing not completely sealed.

Canister not latched.

Blockage Alarm



Alarm Display

Visual message displayed along with audible alarm.

If the alarm is not corrected, continuous audible will occur.



Troubleshooting


Pinch clamps activated.

Drainage tubing kinked.

Address blockage to clear alarms.

Changing Batteries

1. Remove canister.
2. Remove battery cover.
3. Remove batteries and replace them with 3 AA batteries. Lithium batteries are preferred.
4. Replace battery cover.
5. Place the canister back on pump.

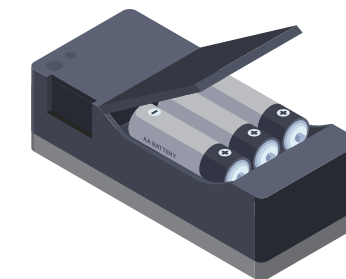

Do not mix alkaline and lithium batteries, also do not mix old and new batteries.

Placing Versa Pump in Carrying Bag

The VERSA NPWT System Bag should always be utilized. The VERSA NPWT System should remain in the carrying bag unless a canister or battery change is necessary. **Steps:**

1. Unzip the top of the bag.
2. Place VERSA NPWT System in bag. The top of the pump should be at the top of the zipper area on the bag.
3. Zip the bag once pump is inserted into bag.
4. Double check correct orientation of pump in bag. The button of the VERSA NPWT System should be at the bottom of the bag.
5. Close front flap on bag.

